



How to Interpret Results

1 Collect 20mls of urine. A minimum of 20mls is required to run a test.


2 Check the temperature strip has a green shade between 32-37°

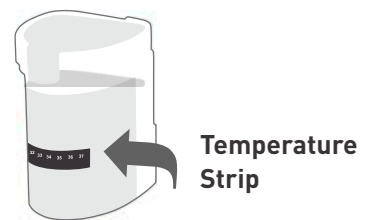


3 Push the key in to activate the test.

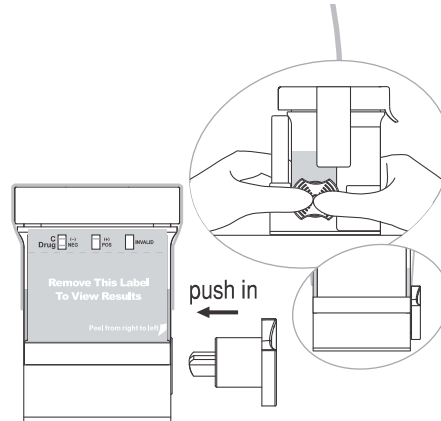

4 Peel off the label to view results. Remove sticker from test panel and discard. Ensure all the control lines (c) are showing on the test panel. If not, the test is invalid.

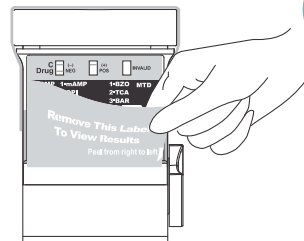


Read drug test results at 5 minutes.

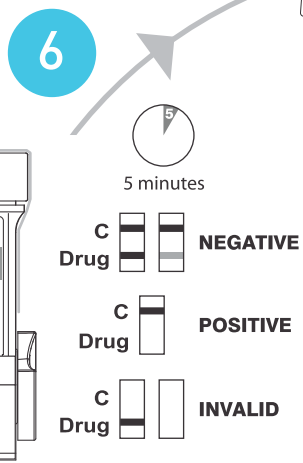

Note: A negative test result (i.e. All lines present) will often appear within the 5 minute time limit, if so the result can be declared.

5 Read the adulteration strip(s) between 3 & 5 minutes. Do not read after 5 minutes

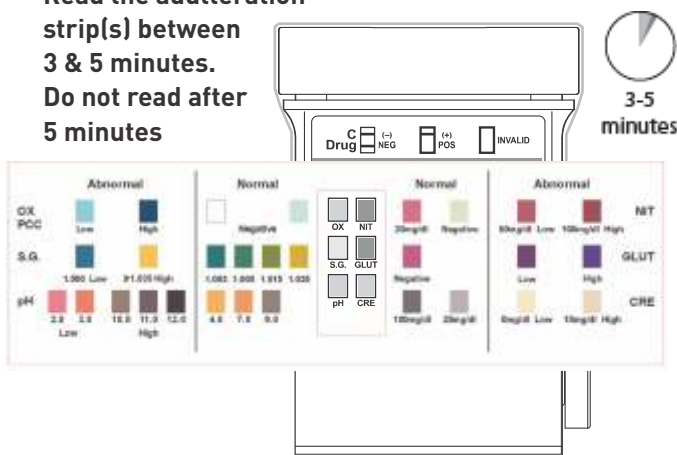

Procedure for Sure Step Cup

step 1

- Remove the cup from foil package.
- Remove key from the cup and open the lid.

step 2

- Collect a minimum of 20mls of urine.
- Check temperature strip - any faint change in colour on the temperature strip indicates that the urine is at body temperature - between 32 - 37 ° C.

step 3

- Push key in to activate test.
- Remove green sticker from test panel and start timing the test.

step 4

- Wait for 3-5 minutes then read adulteration results.

step 5

- Place adulterant colour comparison chart window over the test panel on the cup.
- Compare the colours on the chart to the colours on the cup to ensure that they fall within the normal range.
- If the adulterants are abnormal this indicates the urine has been altered or contaminated and a new sample should be obtained for further testing.

step 6

- Read drug test results at 5 minutes.
- Ensure all the control lines (C) are showing on the test panel, if not the test is invalid.
- A negative test will show a line, even if it is a faint line, this is still a negative.

step 7

- Any positive results will have no line present; these tests should be then sent to a lab for confirmation testing.
-